



ADAPTARAP®

Technical Information:

- ADAPTARAP® naturally dissolves between two hours and five days, depending on moisture level of the environment.
- ADAPTARAP® is biodegradable.
- Each roll wraps 5 to 6 hooves = 15 to 18 wrap applications per pack.
- Recommend ADAPTARAP® in facilities that are actively using footbaths. If no footbath, Moo-wrap continues to be our first wrapping recommendation.

Cold Weather Application:

A small amount of water is required for ADAPTARAP™ to adhere to itself. During cold temperatures, challenges with water freezing and ADAPTARAP® becoming too cold to adhere properly are common.

Tips to improve effectiveness:

- Keep ADAPTARAP® close to your body in a pocket or inside a warm box.
- Use hot water, to wet end of wrap.
- Apply adhesive spray to glue end of ADAPTARAP® to itself.
- Use a heat gun to warm end of wrap to improve its adherence.
- End of ADAPTARAP® can also be “tied off” as with regular wraps.

